Vis de cicatrisation

Gingival former

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Description : The healing abutment allows the soft tissue modeling during the healing phase. It can be used the day of the implant surgery to establish at once the trans-mucous connection or during the second stage, put in deferred loading, after the burying of the implant under the mucous membrane. In that case, a second surgical phase is necessary to discover the implant, remove the cover screw and putting the healing abutment. Available in several heights, H = 2, 4 or 6 mm and several emerging, Ø3.8, 4.8, 5.5 and 6.5 mm to offer better aesthetics results-. To obtain a harmonious and natural minimum outline of gum, a distance of 1.25 mm will be necessary between the edge of the healing abutment and the neighboring teeth. The healing abutment have to exceed 1 mm the gum in the highest place.

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Use : The healing abutment can be used with a bonded bridge or a partial removable prosthesis. The healing period preceding the prosthetic connection can be from weeks to several months.

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Indications : Soft tissue enhancement - Single-unit, partial or total Restoration in the lower or upper maxillary - Temporary Use only- Several forms: cylindrical one piece or anatomic in two parts for premolars guaranteeing a good modeling of tissues. It is possible to use a temporary abutment with a crown in resin during the soft tissue healing period.

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Installation : For the installation, use the 1.2 mm manual (TMH12-C) or Contra-angle (MTH12-C) hexagonal screwdriver with the torque wrench (441-1000235) to tighten the healing abutment.

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